

Treat The
Cause of Dry
Eye Syndrome



Improving the
lives of people
suffering from
**Dry Eye
Syndrome**



TearRestore.com



**TearRestore
EyeRelax**

Mint-Infused Steam
Mask

Mint-Infusion

Mint containing menthol stimulates TRPM8 cold receptors in the eyelid margin, naturally boosting tear production through cool sensing primary afferent neurons.

Studies show that menthol-infused warm compresses can enhance tear meniscus height, tear breakup time, and meibum quality compared to non-infused compresses

Long-Lasting Therapeutic Heat

Delivers over 15 minutes of consistent warmth at an optimal 40°C-43°C, helping to relieve dryness and improve oil flow from the meibomian glands. The sustained heat promotes relaxation while enhancing tear film stability for lasting comfort.


Self-Generating Moist Heat

Featuring air-activated technology, this mask gently generates moist heat without the need for microwaving, making it effortless to use at home, in the office, or before bed. The self-heating design ensures a consistent, comfortable temperature for soothing relief anytime, anywhere.



What Does It Do?

- Delivers Moist Heat at Therapeutic Levels for Over 15 Minutes
- Stimulates Cold Receptors With a Mint Infusion
- Offers a Relaxing Hassle-Free Warm Compress Experience Anytime, Anywhere

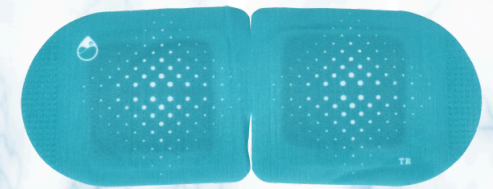

How To Use

Simply open the pouch to activate the self-heating mask, then place it over closed eyes, securing the loops around your ears for a comfortable fit. Relax and enjoy the soothing warmth for at least 15-30 minutes. Ideal for use before bed, while meditating, or anytime you need a moment of relief.